smartdent

smartdent

smartdent

smartdent

smrdntent

smartdent

Instruction For Use

Product Name: smartdental Sterilization Reels
Product Description: smartdental Sterilization Reels are constructed of multilayer PET/PP (Polyethylene Terephthalate / Polypropylene) copolymer film and medical grade paper. A process indicator is applied to the paper surface of the each reel exterior. After exposure to steam and ethylene oxide sterilization, the chemical indicator will provide an accurate visual confirmation by color changing, indicating that sterilization conditions were met.
Intended Use: smartdental Sterilization Reels is a device intended to be used to enclose another medical device that is to be sterilized by a health care provider. It is intended to allow sterilization of the enclosed medical device and also to maintain sterility of the enclosed device until use.

Indicator Color Change: smartdental Sterilization Reels are printed with water based, non-toxic ink to produce Type 1 chemical process indicators. The indicators are in compliance with ISO 11140-1 standard.
Type 1 steam indicator will change color from pink to brown after exposure to pre-vacuum steam sterilization process.
Type 1 ethylene oxide indicator will change from turquoise to yellow after exposure to ethylene oxide sterilization process.

- 1. Do not use if the pack is damaged. Do not reuse. Single use only.
2. One of the open ends of the reel should be closed with a calibrated and validated sealing device to form a pouch.
3. Close the open end of the pouch with a calibrated and validated sealing device. The recommended sealing temperature is 170°C - 190°C and 1-6 second dwell time.
4. Place the item to be sterilized into the pouch. Do not fill pouches to more than 75% of their capacity. Always leave minimum 1.5 cm (0.6 inch) between the material and each seal edge of the pouch for a smooth air flow.
5. Manually remove as much air as possible from the pouch before closing.
6. Close the open end of the pouch with a calibrated and validated sealing device.
7. Seal the pouch horizontally, working on a flat surface.
8. Do not breathe into the sterilizer; place the pouches into the basket in a way that film faces film and paper faces paper.
9. Sterilization pouches should be put upright in the bag or container and not too tight together.
10. Always dry the pouch for pre-vacuum steam sterilization process, minimum 20 minutes.
11. smartdental Sterilization Reels and Pouches have been validated for a minimum drying time of 20 minutes.
12. Handle pouches with care when removing from the sterilizer to prevent tearing. Store pouches in an area protected from sunlight, water and other liquids.
13. Do not use wet pouch. Repeat the sterilization process after changing the pouch.
14. To open the pouch, use a thumb and index finger to hold the paper side and a thumb and index finger to pull back the film side of the pouch. Always open the pouch from the direction of the arrow. For a clean peel open the pouch slowly.

Lot Number: Each smartdental Sterilization Reels is provided with a LOT number printed on the medical grade paper. The LOT number allows the traceability of the product during the manufacturing process. If there is any complaint or feedback, you may store the LOT number and provide picture of it.

Storage Conditions: smartdental Sterilization Reel must be stored in their original packaging at max. +35°C and max. 70% RH humidity. Sterilization packaging should be protected from direct light, moisture and should be stored on the paper side of the pouch is wet, do not use it anymore and dispose it as specified.
 Shelf Life / Expiry Date: The shelf life (pre-sterilization use) of smartdental Reels; are 5 (five) years after manufacture date under recommended storage and handling conditions. The product must be used within 5 years from date of manufacture. The expiry date, as well as the manufacture date is printed on the inner pack label and the shipping carton label. It is intended to allow sterilization of the enclosed medical device and also to maintain sterility of the enclosed device until used. If any serious incident has occurred in relation to the product, user must refer to the manufacturer and has right to report to the competent authority of the member state in which user and/or patient established.

Disposal Instructions: Thrown used or damaged sterilization reels into the autoclavable biohazard bags, not into the regular bags.

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Storage Conditions: smartdental Sterilization Reel must be stored in their original packaging at max. +35°C and max. 70% RH humidity. Sterilization packaging should be protected from direct light, moisture and should be stored on the paper side of the pouch is wet, do not use it anymore and dispose it as specified.
 Shelf Life / Expiry Date: The shelf life (pre-sterilization use) of smartdental Reels; are 5 (five) years after manufacture date under recommended storage and handling conditions. The product must be used within 5 years from date of manufacture. The expiry date, as well as the manufacture date is printed on the inner pack label and the shipping carton label. It is intended to allow sterilization of the enclosed medical device and also to maintain sterility of the enclosed device until used. If any serious incident has occurred in relation to the product, user must refer to the manufacturer and has right to report to the competent authority of the member state in which user and/or patient established.

Disposal Instructions: Thrown used or damaged sterilization reels into the autoclavable biohazard bags, not in the normal rubbish/containers, not in the normal rubbish/containers.

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